



April 22, 2011

Dear Valued Customer,

In general, proficiency survey samples for Influenza A&B testing are in a liquid, "wash" format. As the OSOM Influenza A&B Test does not have a wash procedure, we are updating our recommendations for testing proficiency samples.

For optimal results please use the following procedure:

1. Add a minimum of 135 μ L (maximum of 300 μ L), of flu proficiency sample to one of the test tubes provided in the OSOM Test kit.
2. Add a Test Stick and incubate for 10 minutes.
3. Read results.

NOTE: If needed, 135 μ L disposable pipettes are available from the Sekisui Diagnostics Technical Support Team.

If you have any questions or require additional assistance, please contact a member of our Technical Support Team at (800) 332-1042.

Sekisui Diagnostics, LLC

31 New York Avenue
Framingham, MA 01701
Tel: 800-332-1042 Fax: 508-271-9399