

OSOM[®]

Mono Test

TO AID IN THE DETECTION OF MONONUCLEOSIS



FOR USE WITH WHOLE BLOOD, SERUM, OR PLASMA

FEATURES:

- ✓ **ACCURATE;** >99% sensitivity; 96% specificity
- ✓ **FAST;** Results in 5 minutes
- ✓ **NO AGE LIMITATIONS**
- ✓ **EASY-TO-READ;** Two-color results
- ✓ **WORLD-CLASS TECHNICAL SUPPORT;** Staffed with certified medical technologists and skilled professionals



TEST PROCEDURE / RESULTS



For serum, plasma or whole blood from collection tubes, add 1 drop to test tube from pipette.

For fingertip blood, fill capillary tube and dispense blood into test tube.




Add 1 drop of diluent to test tube. Mix.

Place the test stick into the treated sample.



Read results at 5 MINUTES.

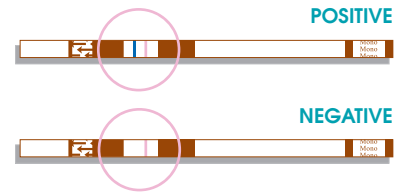

ORDERING INFORMATION

○ OSOM[®] MONO TEST

CONFIGURATION

25 TESTS + 2 TESTS FOR QC

CATALOG NUMBER

145 OR 145E (CE MARKED)



THE AMERICAS
 Sekisui Diagnostics, LLC
 31 New York Avenue
 Framingham, MA 01701
 Phone: 800 332 1042
 Fax: 800 762 6311
 Email: questions@sekisuidiagnostics.com
 ISO 13485:2003 Certified

INTERNATIONAL
 Sekisui Diagnostics (UK) Limited
 50 Gibson Drive
 Kings Hill, West Malling, Kent, ME19 4AF, UK
 Phone: +44 (0) 1732 220022
 Fax: +44 (0) 1732 220024
 Email: info@sekisuidiagnostics.com
 ISO 9001:2000 Certified, ISO 13485:2003 Certified



OSOM[®], the OSOM Training logo and the QC Inside logo are registered U.S. trademarks of Sekisui Diagnostics, LLC. ©2011 Sekisui Diagnostics, LLC – All rights reserved.