

# OSOM<sup>®</sup>

## H. pylori Test



FOR THE DETECTION OF H. PYLORI ANTIBODIES


FOR USE WITH WHOLE BLOOD, SERUM, OR PLASMA

FEATURES:

- ✓ **EASY-TO-USE;** Single reagent procedure fits easily into any practice or lab
- ✓ **FAST;** Results in 10 minutes or less
- ✓ **ACCURATE;** Sensitivity of 95.9% versus biopsy/histology
- ✓ **EASY-TO-READ;** One line indicates negative; two lines indicate positive
- ✓ **SMALL SAMPLE SIZE;** Test only requires 10uL of serum or plasma; 25uL of whole blood
- ✓ **ROOM TEMPERATURE STORAGE**




TEST PROCEDURE / RESULTS

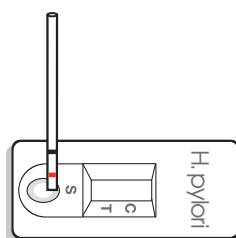

For serum or plasma, fill capillary tube to the red line.

For whole blood, fill to the black line.

Apply sample by lightly tapping the tube on the pad.


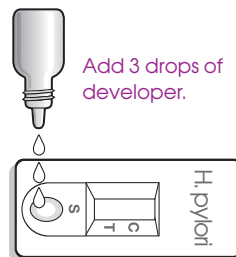


Add 3 drops of developer.

10 MIN



Read at 10 minutes.

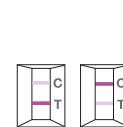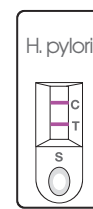

POSITIVE



NEGATIVE



INVALID

ORDERING INFORMATION

- OSOM<sup>®</sup> H. PYLORI TEST
- H. PYLORI CONTROL KIT

CONFIGURATION

25 Tests

CATALOG NUMBER

175 (CE marked)  
176



**THE AMERICAS**  
31 New York Avenue  
Framingham, MA 01701-9322  
Phone: 800 332 1042  
Fax: 800 762 6311  
Email: usdiagcustomerservice@genzyme.com  
ISO 13485:2003 Certified

**INTERNATIONAL**  
50 Gibson Drive  
Kings Hill, West Malling  
Kent, ME19 4AF, UK  
Phone: +44 (0) 1732 220022  
Fax: +44 (0) 1732 220024  
Email: ukdiagcustomerservice@genzyme.com  
ISO 9001:2000 Certified, ISO 13485:2003 Certified



MADE IN THE USA

1208-2 05/10