

RAPID DIAGNOSTICS

# osom<sup>®</sup>

## Influenza A&B Test

FOR THE DETECTION OF INFLUENZA A&B VIRAL NUCLEOPROTEIN ANTIGENS



[www.osomtests.com](http://www.osomtests.com)



FEATURES:

- ✓ **ACCURATE, DIFFERENTIAL DIAGNOSIS;**  
Differentiates between influenza A&B
- ✓ **FAST;** Less than 1 minute hands on time; results in 10 minutes or less
- ✓ **NASAL SWAB SPECIMENS**
- ✓ **QC INSIDE;** Includes 2 additional tests for external QC
- ✓ **TRAINING;** Online training and certification; [www.osomtraining.com](http://www.osomtraining.com)
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologists



# OSOM<sup>®</sup>

## Influenza A&B Test

**Sekisui Diagnostics, LLC (AMERICAS)**  
 4 Hartwell Place Lexington, MA 02421  
 Phone: 800 332 1042  
 Fax: 800 762 6311  
 Email: [osomrapidtests@sekisui-dx.com](mailto:osomrapidtests@sekisui-dx.com)  
 ISO 13485 Certified

**Sekisui Diagnostics (UK) Limited (INTERNATIONAL)**  
 Liphook Way, Allington, Maidstone, Kent ME16 0LQ, UK  
 Phone: +44 1622 607800 Fax: +44 1622 607801  
 Email: [info@sekisui-dx.com](mailto:info@sekisui-dx.com)  
 ISO 9001 Certified, ISO 13485 Certified

**TEST PROCEDURE / RESULTS**




Add extraction buffer to the tube.

Add swab to tube and vigorously mix.




Squeeze side of tube to express liquid.

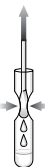

Place the test stick into test tube.



Read results at 10 MINUTES.

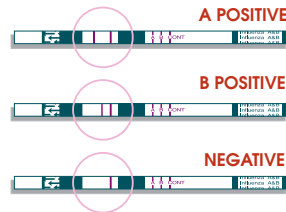

DESCRIPTION	CONFIGURATION	CATALOG #	STORAGE	CPT CODE
OSOM <sup>®</sup> INFLUENZA A&B TEST INFLUENZA A&B CONTROL KIT	25 TESTS+2 TESTS FOR QC	190 OR 190E (CE MARKED) 191 (CE MARKED)	ROOM TEMP ROOM TEMP	87804 (FLU A) 87804-59 (FLU B) IF BOTH ARE ORDERED TOGETHER

